



How to Use The Epi-Pen Auto-Injector

Remove the EpiPen® Auto-Injector from the carrier tube and follow these 2 simple steps:

1



- Hold firmly with orange tip pointing downward.
- Remove blue safety cap by pulling straight up. Do not bend or twist.

2



- Swing and push orange tip firmly into mid-outer thigh until you hear a 'click'.
- Hold on thigh for several seconds.



Built-in needle protection

- When the EpiPen® Auto-injector is removed, the orange needle cover automatically extends to cover the injection needle, ensuring the needle is never exposed.



After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911.

For more information visit [EpiPen.ca](https://www.epipen.ca).